ResMed

AirFit™ P10

for AirMini

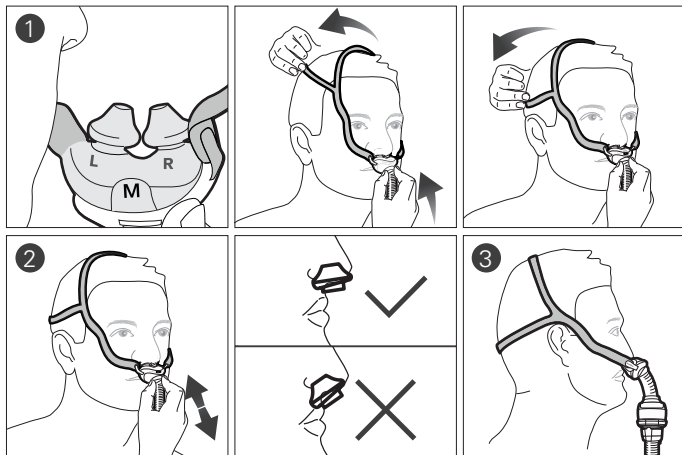
Nasal pillows system


User guide

Svenska


Fitting / Colocación / Anlegen / Mise en place / Applicazione /
Opzetten / Tillpassning / Tilpasning / Sovittaminen / Tilpasning
/ Colocación / Colocação / Τοποθέτηση / Takma / Zakładanie /
Felhelyezés / Fixarea măștii / Установка маски / Paigaldamine /
الارتداء / Nasazení masky / 配戴 / 配戴


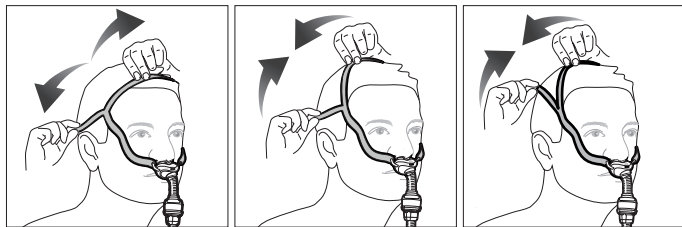
Adjustment / Ajuste / Anpassen / Ajustement / Regolazione /
Afstellen / Justering / Justering / Säätäminen / Justering / Ajuste /
Ajuste / Προσαρμογή / Ayarlama / Regulacja / Beállítás / Reglare /
Регулировка / Kohendamine / الضبط / Úprava masky / 调整 / 調整


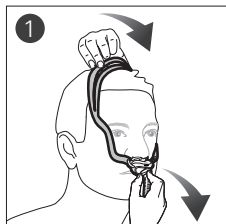
Removal / Remoción / Abnehmen / Retrait / Rimozione / Afnemen
 / Borttagning / Fjerning / Pois ottaminen / Fjernelse / Retirada
 / Remoção / Αφαίρεση / Çıkarma / Zdejmowanie / Eltávolítás /
 Îndepărtare / Снятие маски / Eemaldamine / النزع / Sejmucie masky /
 取下面罩 / 取下面罩



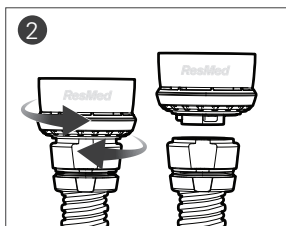
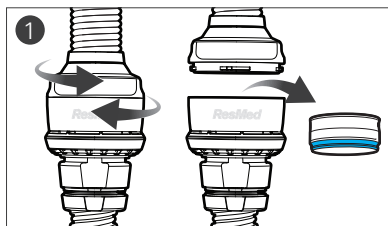
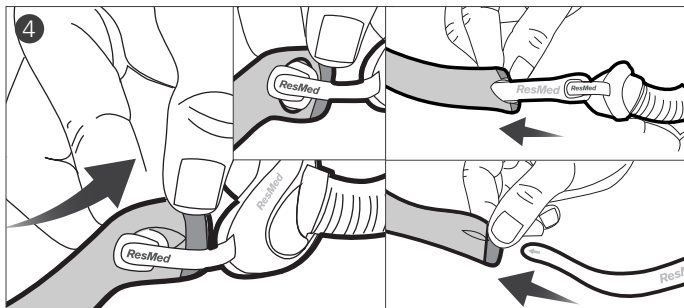
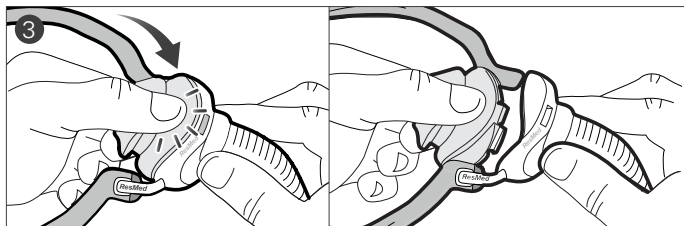
Disassembly / Desmontaje / Auseinanderbau / Démontage
 / Smontaggio / Demonteren / Demontering / Demontering /
 Maskin purkaminen / Adskillelse / Desmontaje / Desmontagem
 / Αποσυναρμολόγηση / Sökme / Demontaž / Szétszerelés /
 Dezasamblare / Разборка / Osadeks lahtivõtmine / التفكيك / Demontáž
 / 拆卸 / 拆卸

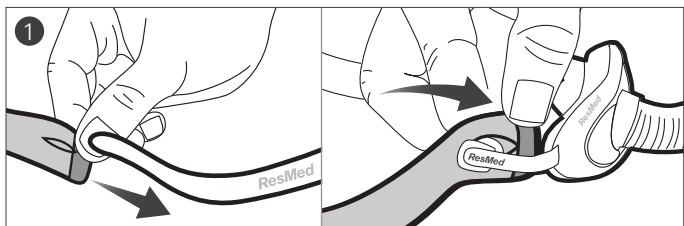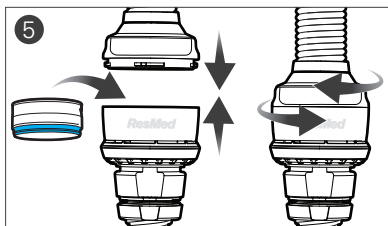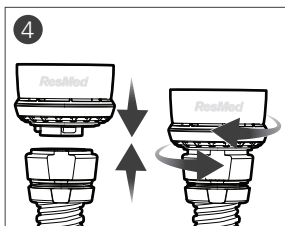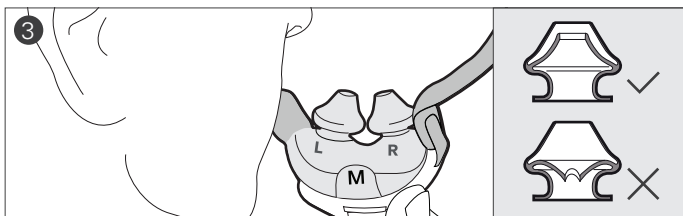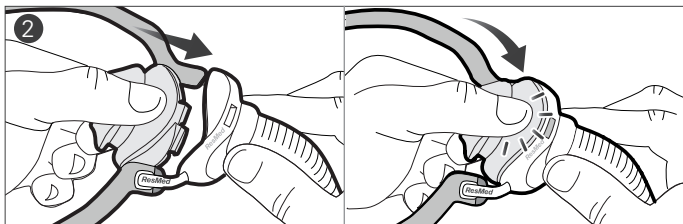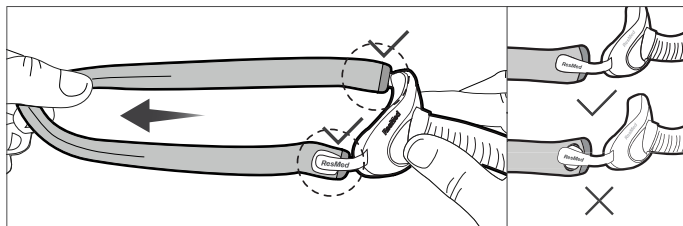
Reassembly / Montaje / Zusammenbau / Remontage /
 Riassemblaggio / Opnieuw monteren / Monteren / Monteren /
 Maskin kokoaminen / Samling / Nuevo montaje / Nova montagem
 / Επανασυναρμολόγηση / Yeniden montaj / Składanie / Ismételt
 összeszerelés / Reasamlare / Повторная сборка / Uuesti
 kokkupanek / إعادة التجميع / Opětovné sestavení / 重新組裝 / 重新組裝

Cleaning the vent / Limpieza del orificio de ventilación /
 Luftauslassöffnung reinigen / Nettoyage de l'orifice de
 ventilation / Pulizia del dispositivo per l'esalazione / Reinigen
 van de ventilatieopening / Rengöra ventilöppningarna / Rengjøre
 ventileringsåpningen / Poistoilma-aukon puhdistaminen /
 Rengøring af lufthullet / Limpieza de los orificios de ventilación /
 Limpeza do respiradouro / Καθαρισμός ανοίγματος εξαερισμού / Hava
 deliğini temizleme / Czyszczenie odpowietrznika / A szellőzőnyílás
 tisztítása / Curățarea aerisirii / Чистка вентиляционного отверстия
 / Ventilatsiooniava puhastamine / تنظيف ثقب التهوية / Čištění
 ventilačního otvoru / 清洗通气孔 / 清洗通氣孔

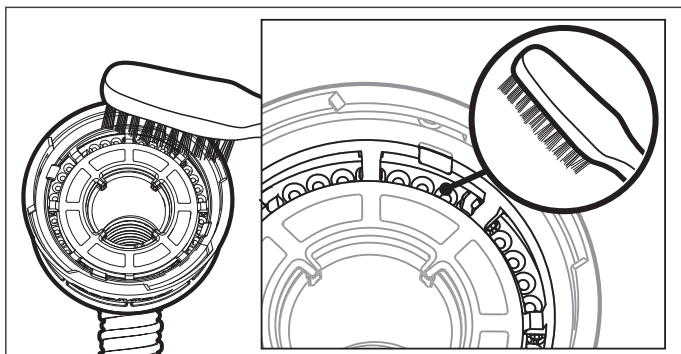for AirMini

Näskuddmask

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E AirFit P10 for AirMini frame / Armazón de la AirFit P10 for AirMini / AirFit P10 for AirMini Rahmen / Entourage rigide AirFit P10 for AirMini / Telaio AirFit P10 for AirMini / AirFit P10 for AirMini frame / AirFit P10 for AirMini maskram / AirFit P10 for AirMini ramme / AirFit P10 for AirMini -runko-osa / AirFit P10 for AirMini-ramme / Armazón de la AirFit P10 for AirMini / Armação da AirFit P10 for AirMini / Πλαίσιο AirFit P10 for AirMini / AirFit P10 for AirMini çerçeve / Ramka nośna maski AirFit P10 for AirMini / AirFit P10 for AirMini keret / Cadru AirFit P10 for AirMini / Κορυς AirFit P10 for AirMini / Maski AirFit P10 for AirMini raam / راطل AirFit P10 for AirMini / Tělo masky AirFit P10 for AirMini / AirFit P10 for AirMini 面罩框 / AirFit P10 for AirMini 面罩框

6 Frame arms / Brazos del armazón / Rahmenstützen / Bras de l'entourage rigide / Bracci del telaio / Framearmen / Ramfästen / Rammearmer / Rungon sivukappaleet / Rammearme / Brazos del armazón / Braços da armação / Βραχίονες πλαισίου / Çerçeve kolları / Ramiona ramki / Keretmerevítők / Brațele cadrului / Планки корпуса / Raami toed / راطل اعارد / Ramena tēla masky / 面罩框支撐臂 / 面罩框支撐臂

7 Node / Saliente / Anschluss / Nœud / Nodo / Knop / Fäste / Node / Kiinnitysnystyrä / Knop / Enganche / Encaixe / Κόμβος / Düğüm / Występ / Rögzítőelem / Nod / Пряжка / Sölm / متقعال / Výstupek / 节点 / 節點

8 Mask Tube / Tubo de la mascarilla / Maskenschlauch / Circuit du masque / Tubo della maschera / Maskerslang / Maskslang / Maskeslange / Maskista lähtevä letku / Maskeslange / Tubo de la mascarilla / Tubagem da máscara / Σωλήνας μάσκας / Maske Hortumu / Rura maski / Maszk csöve / Tubul măștii / Трубка маски / Maski voolik / عانقل البوبن / Trubice masky / 面罩管路 / 面罩管路

9	Vent module / Módulo de ventilación / Ausatemöffnungsmoduls / Module de ventilation / Modulo del dispositivo per l'esalazione / Ventilatiemodule / Ventilmodul / Ventilasjonsmodul / Poistoilmamoduuli / Udluftningsmodul / Módulo de ventilación / Módulo do respiradouro / Μονάδα εξαερισμού / Hava deliği modülü / Moduł odpowietrzający / Szellőzőegység / Modul de aerisire / Вентиляционный модуль / Ventilatsioonimoodul / قندحو / قندحو متلا / Ventilací modul / 通气孔模块 / 通氣孔模組	
10	Air tubing / Tubo de aire / AirMini-Schlauchsystem / Circuit respiratoire / Circuito respiratorio / Luchtslang / Luftslang / Luftslange / Hengityslakku / Luftslange / Tubo de aire / Tubagem de ar / Σωλήνωση αέρα / Hava tüpü / Rura przewodząca powietrze / Levegőt vezető cső / Tuburi de aer / Дыхательный шланг / Õhuvoolik / عاؤملا بوبنأ / Vzduchová trubice / 呼吸管路 / 呼吸管路	
F	HumidX (pk / Paqu. de u / Stk. / Paquet de / Czf da / st. / -pack / -pakning / kpl / Pakke med stk. / Paquete de / unid. / ανά συσκευασία / pk / Opakowanie szt. / darabos csomag / buc. / шт/упак / Pakis / تادحو / قبلع / Balení po ks / 套 / 套)	38809 (3pk) 38810 (6pk)
	HumidX Plus (pk / Paqu. de u / Stk. / Paquet de / Czf da / st. / -pack / -pakning / kpl / Pakke med stk. / Paquete de / unid. / ανά συσκευασία / pk / Opakowanie szt. / darabos csomag / buc. / шт/упак / Pakis / قبلع / تادحو / Balení po ks / 套 / 套)	38812 (3pk) 38813 (6pk)

XS: Extra small / Extrapequeña / Extra Small / Extra small / Extra Small / Extra small / Extra small / Extra small / Erittäin pieni / Extra small / Extrapequeña / Extra pequeno / Πολύ μικρό / Ekstra küçük / Rozmiar bardzo mały / Extra kicsi méret / Foarte Mică / Очень маленький / Eriti väike / ادج ري غص / extra malá / 加小号 / 加小號

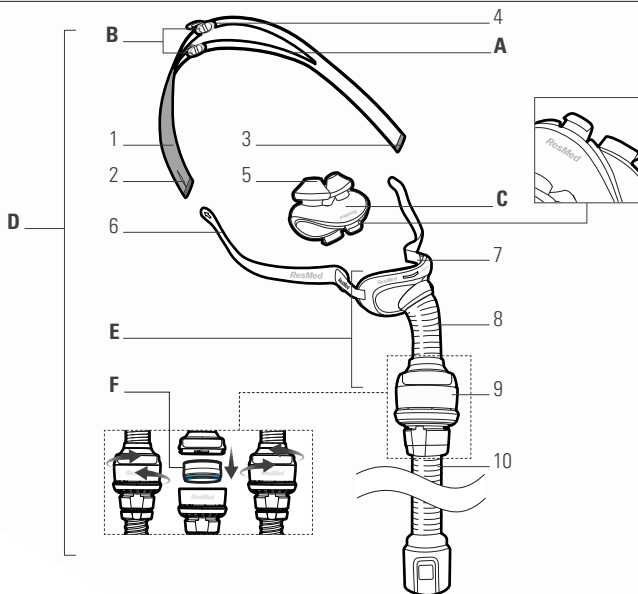
- S: Small / Pequeña / Small / Small / Small / Small / Small / Small / Pieni / Small / Pequeña / Pequeno / Μικρό / Küçük / Rozmiar mały / Kicsi méret / Mică / Маленький / Väike / ري غص / malá / 小号 / 小號
- M: Medium / Mediana / Medium / Medium / Medium / Medium / Medium / Medium / Keskikokoinen / Medium / Mediana / Médio / Μεσαίο / Orta / Rozmiar średni / Közepes méret / Medie / Средний / Keskmine / طسو / střední / 中号 / 中號
- L: Large / Grande / Large / Large / Large / Large / Large / Large / Suuri / Large / Grande / Grande / Μεγάλο / Büyük / Rozmiar duży / Nagy méret / Mare / Большой / Suur / ري بك / velká / 大号 / 大號

ResMed

AirFit™ P10

for AirMini

ResMed Ltd
1 Elizabeth Macarthur
Drive Bella Vista NSW
2153 Australia

Distributed by
ResMed Corp 9001 Spectrum Center Boulevard San Diego
CA 92123 USA **EC REP** ResMed (UK) Ltd 96 Jubilee Ave Milton Park
Abingdon Oxfordshire OX14 4RW UK

See ResMed.com for other ResMed locations worldwide. AirFit, AirMini and HumidX are trademarks and/or registered trademarks of the ResMed family of companies. For patent and other intellectual property information, see ResMed.com/ip. © 2017 ResMed Ltd. 628124/1 2017-02

CE
0123

ResMed.com

628124